

Madsen Alpha Quick Use Guide



To Screen – 2 button pushes!

1. Press “On” button to turn on
2. Press the red ear graphic for right or the left ear graphic for left and the test will run

Other Quick functions:

- To screen the other ear or run another test – press the other ear graphic that will show up at the bottom of the test result screen (automatically defaults to the one you did not screen)
- Press the arrow button at the bottom of the test result screen to retest the same ear

Power

- Has an “on and off” button. Push to turn on. You can turn it off after use OR it will go to a power saving and then turn itself off after a period of non use.
- 8 hours of continuous use with fully charged battery

PE Tubes -

- You can test children with known PE tubes and should expect a pass if the ears are clear and PE tubes are functioning.

Maintenance

- Charge battery at room temperature to increase battery life
- Recommended to replace the rechargeable battery every 12 – 18 months
- Annual calibration - Complete annually to ensure accuracy and to obtain software updates etc.
- Probe tips should be replaced when they become clogged. Do not attempt to clean them - they are disposable and must be replaced when they become clogged.
- Cleaning:
 1. May be wiped clean with moist tissue with alcohol to clean the surface of the device
 2. Do not put excessive pressure on the clear display window or allow any utensil to puncture the display window or keypad.
 3. Do not allow any fluid to enter the device or immerse the instrument in fluids or attempt to sterilize the instrument or any or its accessories.

