

Otodynamics Otocheck Screener Quick Use Guide



To Screen - 3 Button pushes!

1. Press the Green button (lower left). It will ask you if you want to turn the unit on.
2. Press the green “dot” button on upper right (just under the screen) under the “Yes” (Press it quickly or it goes off).
3. Wait for the test screen to come on – it has an ear graphic. Push the green dot button again the screening will begin. Remember “green” for go! This will always be your start-screening button.

Other Quick functions:

- SAVE a result: You choose the ear screened when you have completed the screening for that ear and you are saving the test (use if you need to save or print results).
- PAUSE or STOP: To pause or stop testing - press the button on the left under the screen with the blue square on it (such as if child is too noisy - then quiets).
- OFF: To turn the screening unit off, hold the power button down. The screen will ask you if you want to turn it off – press the button under the yes to turn it off.

Power:

- 3 to 4 hours to fully charge the re-chargeable battery
- 8 hours of testing – the screening unit will screen for approximately 8 hours of continuous testing before it needs re-charged
- You **cannot** test with the unit plugged into a power outlet while it re-charges.

PE Tubes -

- You can test children with known PE tubes and should expect a pass if the ears are clear and PE tubes are functioning.

Maintenance

- Annual calibration - Complete annually to ensure accuracy and to obtain software updates etc.
- Probe tips should be replaced when they become clogged Do not attempt to clean them - they are disposable and must be replaced when they become clogged.
- Cleaning:
 1. May be wiped clean with a damp cloth using a mild antiseptic solution (e.g. cetylalide).
 2. Do not put excessive pressure on the clear display window or allow any utensil to puncture the display window or keypad.
 3. Do not allow any fluid to enter the device or immerse the instrument in fluids or attempt to sterilize the instrument or any of its accessories.

