

Original Maico Ero*Scan & OtoRead - Discontinued Quick Use Guide



To Screen - 2 Button pushes!

1. Press large button (with down arrow) to turn on
2. Press R or L smaller buttons to start testing

Other Quick functions:

- RE-START: To re-start a test, press the large button during testing to retest (such as if child is too noisy - then quiets)
- STOP: To stop or back out of testing - press small center button with up arrow to stop testing
- OFF : To turn off, hold small center button down until it turns off

Front screen lights (3) mean:

- Left light is an error message meaning something is not allowing the test to work appropriately such as the environment is too noisy
- Center lights up (amber) when testing, I
- Right light is green indication machine is ready to test

PE Tubes -

- Children with known PE tubes can be tested by bypassing the auto-start function of the unit.
Do this by:
 - First inserting the probe in the ear canal
 - Disable the auto-start by selecting the ear to be tested by holding down the right or left arrow keys for 3 seconds until the green "test" light turns off.
 - Release holding the key - the Ero*Scan/OtoRead will calibrate and test as before.

Maintenance

- Annual calibration - Complete annually to ensure accuracy and to obtain software updates etc.
- Use AA batteries - do not use rechargeable batteries (uses AA batteries with a life of about 300 tests). Write down dates of when you change batteries and keep spares on hand.
- Probe tips should be replaced when they become clogged Do not attempt to clean them - they are disposable and must be replaced when they become clogged.
- Cleaning:
 1. May be wiped clean with a damp cloth using a mild antiseptic solution (e.g. cetylaldehyde).
 2. Do not put excessive pressure on the clear display window or allow any utensil to puncture the display window or keypad.
 3. Do not allow any fluid to enter the device or immerse the instrument in fluids or attempt to sterilize the instrument or any of its accessories.

