

# EQUIPMENT SET-UP

## Quick Reference Guide

The equipment must be unpacked and all cabling connected to the laptop and Duet prior to the infant's arrival.

Duet



Back of Duet

All of the cords connecting to the Duet are labeled with the numbers 1-5. Match the number on the cord to the number on the Duet.

1. Connect the following to the Duet:

- a. Power cord #1
- b. USB from laptop #2
- c. Insert earphones #3
- d. Bone oscillator #4
- e. Electrodes #5

2. Connect the following to the laptop:

- a. USB to Duet cable (#2)
- b. Mouse
- c. Power cord
- d. Video otoscopy USB

3. Connect the following to a wall outlet or power source:

- a. Duet power supply
- b. Laptop power supply



Electrodes

Insert  
Earphones

Bone  
Oscillator

